Technische Daten des Gehörschutzes ePRO-X.S nach EN 352-2:2002
Technical data of the hearing protection ePRO-X.S in accordance with EN 352-2:2002
Données techniques de la protection auditive ePRO-X.S selon la norme EN 352-2:2002
Dati tecnici dell'otoprotettore ePRO-X.S conforme a EN 352-2:2002
Datos técnicos de la protección auditiva ePRO-X.S conforme a EN 352-2:2002

	Hz	63	125	250	500	1000	2000	4000	8000
ePRO-X.S X.5M ○ ○ ○ ○ ○	Mf	26,8	27,8	27,5	28	28,5	34,4	38,2	34,8
	Sf	3,1	3,6	4,2	3,4	3	4	4,7	5,6
	APV	23,7	24,2	23,3	24,6	25,5	30,4	33,5	29,2
	SNR: 30 – L: 25, M: 26, H: 30 – W, S, V, E ₁								
ePRO-X.S X.4M ○ ○ ○ ○ ○	Mf	24,6	25,2	25,2	24,6	25,5	32,3	36,7	31,8
	Sf	3,9	4,9	3	3,2	3,6	2,8	4,7	6,9
	APV	20,7	20,3	22,2	21,4	21,9	29,5	32	24,9
	SNR: 27 – L: 22, M: 23, H: 27 – W, S, E ₁								
ePRO-X.S X.3M ○ ○ ○ ○	Mf	24	25,4	24,2	24,3	25,5	31	35	29,9
	Sf	4,8	3,9	4,2	4,9	4,8	3	3,8	4,3
	APV	19,2	21,5	20	19,4	20,7	28	31,2	25,6
	SNR: 25 – L: 21, M: 22, H: 27 – W, X, S, V, E ₁								
ePRO-X.S X.2M ○ ○	Mf	15,7	18,1	19,2	20,3	22,5	28,9	33	28,1
	Sf	3,9	4,5	2,9	3,5	2,8	4,2	3,5	4,9
	APV	11,8	13,6	16,3	16,8	19,7	24,7	29,5	23,2
	SNR: 23 – L: 17, M: 20, H: 25 – W								

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User instructions for hearing protection ePRO-X.S
With your new, custom-made hearing protection ePRO-X.S you have decided to protect your hearing against damaging or disturbing noise by using custom-made protectors. Your hearing protection device has been manufactured to fit your ear perfectly and is guaranteed to offer you the highest possible wearing comfort. The hearing protector is worn in the ear. It can be inserted with minimal effort, is highly compatible with other protective equipment (e.g. protective glasses) and above all, is appropriate for long stays in noise zones. Properly used, it protects you from hazardous noise. No other use is approved and is therefore not allowed. We hope you enjoy it thoroughly!

Prior to using your hearing protection device
Please read these instructions carefully to familiarize yourself with the proper handling of the hearing protection device. Only if the following suggestions are adhered to can effective protection and the durability of your hearing protectors be guaranteed. Keep these instructions in a safe place.

- Standard features of your hearing protection device**
- Hearing protection earmold (Note color marks: right ear/red, left ear/blue)
 - Attenuation element for the hearing protectors (1 pair)
 - Wax pin (use the metal loop to remove cerumen deposits from the hearing protectors)
 - Sealing plug for additional canal (incl. spare sealing plug)
 - Case
 - User instructions

- Additional features of your hearing protection device**
- Connecting cord (detachable connection between the right and left hearing protector)
 - Safety clip to attach connecting cord to clothes
 - Care gel to ease inserting the protectors
 - Antimicrobial coating
 - Attenuation elements with different attenuation effect - "Technical data" (see table)
 - Metal detection ball

Your hearing protection device is equipped with a sound canal, which contains an attenuation element. Depending on your requirements, you may use other attenuation elements, which can be purchased from your specialist. Moreover, you will find the hearing protectors equipped with an additional canal, checking the effectiveness/perfect fit of your hearing protection device and, in addition, making the use of further additional products possible, e.g. a headset for telephone and radio communication or an earphone for MP3 players.

- Inserting and removing your hearing protection device** (see figure 2 to 4)
1. With clean hands, take the hearing protection ePRO-X.S out of the packaging.
 2. Make sure that your entire hearing protectors have not been damaged and are clean. Do not insert a damaged hearing protector in your ear; in case of damage, contact your specialist.
 3. Note the red and blue color marks on your hearing protectors:
red = protector for the right ear; blue = protector for the left ear.

4. Take the right protector into your right hand, the left protector into your left hand. Be sure to grasp the protector from the bottom with your thumb and from the top with your index finger (figure 2).
5. Take the hearing protector to your ear without twisting it. Angling it from behind, insert it into the auditory canal (figure 3). Be sure to insert the hearing protection devices deeply enough and position it comfortably (figure 4).
6. To facilitate the insertion of the protectors, we recommend using care gel (see additional products).
7. To remove the hearing protector grasp the handle with your thumb and index finger and pull the protector out of your ear. Alternatively you can free the hearing protector by lifting it slightly with your thumb and then removing it from your ear with your thumb and index finger (see figure 2).

- WARNING**
- A) Noise impact on hearing without hearing protection
Even a brief exposure to a high noise level while wearing improperly inserted hearing protectors or no hearing protection devices at all may damage your hearing.
- B) Protective effect that is too low when the hearing protector is incorrectly inserted in the ear
Only expose yourself to high noise levels while wearing your custom-made hearing protectors after you are familiar with handling the protectors and inserting them properly into the auditory canal. Have the correct use shown to you e.g. by the manufacturer or the supplier of the hearing protection earmold or your physician. It may be helpful for beginners to practice the correct technique in front of a mirror.
- C) Handling and daily checks
The additional canal must be sealed with the original sealing plug and should only be removed for use in combination with specific additional products. If the sealing plug is missing, do not wear the hearing protection devices in extreme noise levels! Please check your hearing protector for this on a daily basis.
- D) Leaking/reduced protective effect
Anatomical changes in the shape of the ear may lead to the deterioration of secure earmold fit, e.g. when there is a change in weight or there have been ear operations. Safe protection is only achieved when a specialist has tested the hearing protection earmold after it is delivered and then tests it again at regular intervals of at most three years.
- E) Reduced perception
Please consider: While wearing the hearing protection devices, the signals in the surrounding area will also be muffled (e.g. warning signals). In addition, be sure to adhere to the regulations regarding the use of hearing protection devices in road traffic/track superstructure/railway operation.
- F) Risk of injury with improper handling
Do not pull the hearing protectors out of the ear by the connecting cord. Sudden or fast removal of the ear plugs out of the auditory canal may damage the ear drum.
- G) Risk of the hearing protector being pulled out
These ear plugs should not be used where there is a risk that the connecting cord could be caught up during use. We recommend detaching the connecting cord with the connecting fasteners from the hearing protection device. After the connecting cord has been detached, it can only be reattached by inserting it again (note the connecting fasteners in figure 5).
- H) Connecting additional equipment
As a hearing protection device the ePRO-X.S may only be used in connection with additional products (e.g. attenuation element, headset etc.) that have been type-tested with ePRO-X.S. More information is available from your specialist or the manufacturer.
- I) Skin reaction
The hearing protector is exclusively manufactured from materials that have been proven

- over the years, tested and that are bio-compatible. If you were nonetheless to notice symptoms, remove the hearing protector immediately and inform your doctor, specialist consultant or supervisors.
- J) For small children there is a swallowing hazard!
The product must be kept out of reach of small children.
- K) Ageing and wear
Custom made earmolds and attenuation elements are designed for daily use. However, we recommend having a new spare earmold made after at most 6 years. If necessary, the attenuation element may continue to be used after successful function testing.

Storage and care
This product may suffer if certain chemical substances are used. Your hearing protectors must be cleaned regularly to avoid a decrease in its protective effectiveness, or the development of skin irritations or other ear problems. Use either special cleaning products offered by the manufacturer or at least water and soap. Dry the hearing protectors thoroughly before inserting them again. Do not use chemical solvents, abrasive cleansers or brushes. The attenuation elements must be cleaned professionally. More information is available from your hearing protection professional or the manufacturer. Store the ePRO-X.S in the case which is included in delivery.

ATTENTION
These products may not be altered or modified. If the suggestions in these instructions are not adhered to, the effectiveness of the hearing protection device may be affected considerably. Only by handling the hearing protectors carefully and wearing the hearing protection ePRO-X.S continuously in extreme noise levels, are you protecting your hearing as best as possible. Therefore, we urgently recommend using and maintaining the hearing protection ePRO-X.S

according to these suggestions.

CE Declaration of Conformity
AUDIA AKUSTIK GMBH hereby declares that the hearing protection ear mould ePRO-X.S complies with PSA Regulation (EU) 2016/425.

The complete declaration of conformity can be viewed on our website at: www.audia-akustik.de (Download area)

Relevant notified body for conformity assessment and product monitoring:
PZT GmbH
Bismarckstraße 264 B
26389 Wilhelmshaven | Germany
Identification number: 1974

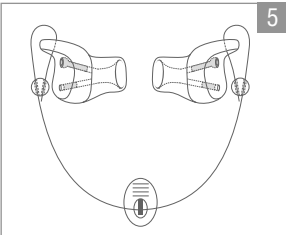
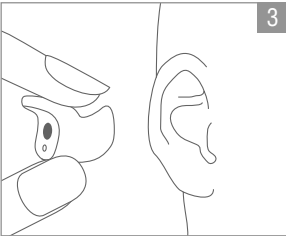
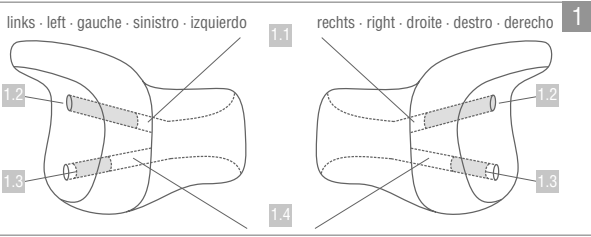
Individuell angepaßter Gehörschutz
Custom-made hearing protectors
Protection acoustique sur mesure
Otoprotettore su misura
Protección auditiva a medida

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egger Otoplastik + Labortechnik GmbH, Germany