Wir hören uns.

**AUDIA**

# **AUDIA ELACIN** ER25 | ER15 | ER09

**EN** | User Manual

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Custom-made hearing protection

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<b>EN</b>	<b>f in Hz</b>	Frequency in Hertz.
	<b>H in dB</b>	Attenuation value in predominantly high-frequency noise spectrum ( $f > 2000$ Hz).
	<b>M in dB</b>	Attenuation value in predominantly medium-frequency noise spectrum ( $f > 500$ and $f < 2000$ Hz).
	<b>L in dB</b>	Attenuation value in predominantly low-frequency noise spectrum ( $f < 500$ Hz).
	<b>SNR in dB</b>	Single Number Rating, to be deducted from C-weighted noise level.
	<b>Mf in dB</b>	Average attenuation value at given frequency $f$ in decibel.
	<b>Sf in dB</b>	Standard deviation at given frequency $f$ in decibel.
	<b>APVf in dB</b>	Assumed Protection Value ( $Mf - 1x Sf$ ), at given frequency $f$ in decibel.

Specifications: The European type certification inspection was performed according: EN352-2:2020+A1:2024 by IFA, 53757 Sankt Augustin, Germany, Kenn.no.: 0121.

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## AUDIA ELACIN ER CUSTOM-MADE HEARING PROTECTION

**IMPORTANT:** Read this instruction carefully before use and keep both the package and the user manual.

### General

The custom-made hearing protection ear mould AUDIA Elacin ER protects your hearing from harmful noise emissions whilst being comfortable to wear. It is intended for a service life of at least two years.

### Use

Check that the hearing protectors are clean before you use them. If necessary use an ear wax remover<sup>†</sup> to clean ear wax out of the sound tube of the hearing protector. Note the red and blue color marks on your hearing protectors: red = protector for the right ear; blue = protector for the left ear. Your hearing specialist will be glad to demonstrate the correct way to insert and remove the hearing protectors.

<sup>†</sup> An ear wax remover is available separately. Ask your hearing specialist for more information.

**Insertion**

Insertion may be less smooth at first use. Application of ear cream on the part of the hearing protector that goes into the ear will improve ease of insertion. See picture on page 4.

**Removal**

Remove the hearing protector using a rotary forward motion. OR, alternatively, place your index finger against the lower part of your ear and press upwards a number of times. The earpiece is then slowly released from the ear. When it no longer seals you can safely remove it.

**Care and maintenance**

Your hearing protectors must be regularly cleaned and inspected to avoid reduced protection, skin irritation or other ear problems. Clean your AUDIA Elacin ER hearing protectors regularly with a damp cloth. For a regular and thorough cleaning of your custom-made hearing protectors, we advise you to use the cleaning products of the manufacturer. These cleaning products help you to keep your hearing protectors in the best possible shape and prolong the life-span. For further information, please contact your hearing protection representative or your hearing care professional. Store the ear protection device in the packaging provided when not in use.

## **Warning**

- Use AUDIA Elacin ER hearing protectors carefully in accordance with the instructions. If the instructions above are not followed, the protection certainty of the hearing protectors cannot be guaranteed.
- Only enter noisy areas with your individually-fitted hearing protectors once you are familiar with their use and have ensured they are defect-free, as well as confirming they fit correctly in your ear canal. Ensure that you have been shown how to insert it correctly, e.g. by the manufacturer or supplier of the hearing protection ear mould or your company doctor. At the beginning, it can be easier to practice this using a mirror.
- Use your AUDIA Elacin ER hearing protectors always in noisy situations! NOTE: the protection is reduced substantially even if you only remove them for a very short period. You will not be adequately protected if you fail to wear the hearing protectors at all times.
- This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.
- If applicable: These hearing protectors should not be used where there is a risk that the connecting cord could be caught up during use, for example, by a machine.
- The hearing protection device must not be pulled out of the ear canal by the optional cord, as this may damage your hearing protection device. To remove the device only hold on to the protruding material at the ear canal entrance.
- Sudden or fast removal of the hearing protectors out of the ear canal may damage the ear drum.

- The hearing protectors are custom-made and are therefore not transferable. The hearing protectors are designed to be inserted in the hearing canal only.
- Proper use and adherence to maintenance and care instructions contribute to the longevity of the product.
- The greatest possible care is taken in the manufacturing of your AUDIA Elacin ER hearing protectors. Immediately contact your supplier if you notice anything out of the ordinary.
- Anatomical changes in the ear shape can negatively affect the seal of the ear moulds, e.g. in cases of weight change or ear surgery. Safe protection is only ensured if the hearing protection device ear mould is checked after delivery and thereafter at regular intervals of a maximum of three years.
- Please bear in mind that, when hearing protection is used, you will also hear signals in your environment more quietly (e.g. warning signals and voice signals). In addition, the regulations on the use of hearing protection in road traffic/track superstructure must be observed.
- The hearing protection device is made exclusively from biocompatible materials that have been proven and tested over a number of years. However, if you do notice any issues, remove the hearing protection device immediately and inform your doctor, specialist advisor or supervisor.
- Choking hazard for infants! The product, its components and packaging must be kept out of the reach of children.
- The product may not be modified or changed.

### **Declaration of conformity**

AUDIA AKUSTIK GMBH hereby declares that the hearing protection ear mould AUDIA Elacin ER complies with PSA Regulation (EU) 2016/425.

The complete declaration of conformity can be viewed on our website at:  
[www.audia-akustik.de/en/service/media-center](http://www.audia-akustik.de/en/service/media-center)

Relevant notified body for conformity assessment and product monitoring:  
PZT GmbH  
Bismarckstraße 264 B  
26389 Wilhelmshaven | Germany  
Identification number: 1974

You can obtain further technical information from the following address:  
AUDIA AKUSTIK GMBH  
Franz-Mehring-Straße 13 | 99610 Sömmerda | Germany  
+ 49 (0) 3634 693-0 | [info@audia-akustik.de](mailto:info@audia-akustik.de) | [www.audia-akustik.de](http://www.audia-akustik.de)

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Elacin is a registered trademark
**ELACIN INTERNATIONAL B.V.**
of Elacin International B.V.